

## COLD HYDROGEL PATCH

### For muscular pain & traumas



#### PRODUCT DESCRIPTION

**COLD HYDROGEL PATCH** provides cooling on pain areas. Fast cooling effect can relieve the instant joint and muscle pain effectively. The cooling effect of the patch lasts for long time. This patch maintains moisturizing and cooling effects to the affected area as a hydrogel patch.

Indicated for strains, sprains, joints, and muscles pain relief.

#### ACTIVE INGREDIENTS

The cooling effect is due to the hydrogel technology and menthol. In contact with the skin, the hydrogel produces a refreshing action that reduces the skin temperature, offering a soothing effect on the muscular tissue.

**L-MENTHOL** is a natural substance analgesic and anti-inflammatory.

#### INSTRUCTIONS FOR USE

- 1- Clean and dry the area for application.
- 2- Cut the sachet and take out the cold hydrogel patch.
- 3- Peel off the protective film and apply patches to the affected area once or twice a day (the dosage may be increased according to patient's symptoms).
- 4- Reseal zip-lock well for storage remained pad in the pouch to prevent drying. Use only once.

Recommended to use once or twice a day. Number of patches may increase based on the patient's symptoms.

The cooling effect of the patch lasts for a long time, up to 8 hours.

#### PRESENTATION

3 patches/pouch. Patch size rectangular shape [8x12 cm].

#### WARNINGS

For external use only. Do not apply around eyes, mouth, nose, mucous, wounds and damaged skin. Discontinue use the patch right away if you get any skin adverse effects including (rash, itching, redness...).

Store in a cool, dry place and air-tight container at controlled room temperature.

#### PRODUCT CATEGORY

Medical Device class I.