

## HYDROGEL BREAST PADS



### PRODUCT DESCRIPTION

The **Breast Hydrogel Pads** are an advanced treatment for broken skin, creating the optimal condition for healing broken skin to restore the skin's natural level of moisture.

The innovative pads have been created with the purpose of accelerating the healing process and soothing tender skin to allow to breastfeeding in comfort .

### INDICATIONS

The hydrogel breast pads are made with a hydrogel supported by breathable non-woven fabric and thanks to this special design the hydrogel breast discs:

- Soothe sore and creaked nipples
- Create the optimal condition for healing broken skin
- Restore the skin's natural moisture and keep it supple
- Cool irritated skin
- Absorb leakage, keeping skin dry and reduce the build-up of bacteria
- Safe to continue breastfeeding while use the discs



### INSTRUCTIONS FOR USE

1. Wash your hands and nipples with lukewarm water before application.
2. Open sterile packet and remove the protective backing from the discs and gently place the disc on the nipple and areola.
3. Before breastfeeding, remove discs from nipples and wash them with lukewarm water.
4. Whilst breastfeeding, place the discs on a clean surface. Discs can be placed in the refrigerator for added cooling comfort.
5. Wash and dry nipples before reapplying discs.
6. Change the discs once a day. In the case of excessive milk leakage, replace discs more frequently.

### PRESENTATION

Individual discs packaged in Pet/Alu sachets. The product is gamma irradiated.

### CATEGORY

Medical Device class IIb.