


PROTECTIVE PREPARATION FOR VESICAL AND URETRAL MUCOSA

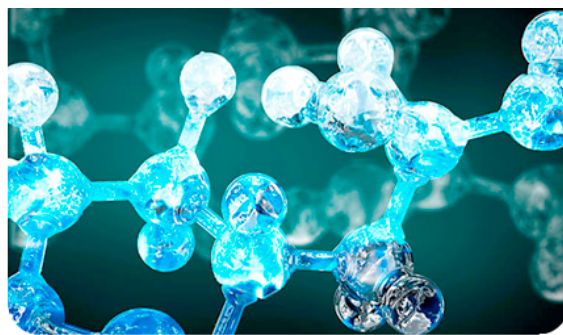

PRODUCT DESCRIPTION

Protective preparation for vesical and urethral mucosa is a sterile medical device, an evolved application of **sodium hyaluronate, both high and low molecular weight**. This polysaccharide is already contained in human body tissues, where it carries out several important biological functions thanks to its film-forming and moisturizing properties, as promoting a more natural and safe cellular regeneration process in the event of vesical and urethral mucosa alterations.

INDICATIONS

Protective preparation for vesical and urethral mucosa is indicated both for prevention, to maintain vesical and urethral mucosa integrity, and for protection, to indirectly promote the physiologic restoration of mucosa lesions.

The product is indicated:

- during radiotherapy, both as prophylaxis and as co-adjuvant for the restoration of vesical and urethral mucosa in patients subject to radiations
- in Urology, for damaged vesical and urethral mucosa
- in Gynecology, for invasive gynecological examinations
- before or after surgery, for vesical and urethral mucosa

MECHANISM OF ACTION

Restoration of the integrity of vesical and urethral epithelium, using Sodium Hyaluronate, carrying out a protective action, thanks to its film-forming properties.

THE ACTIVE INGREDIENTS

High and low hyaluronic acid salt.


INSTRUCTIONS FOR USE

Wash hands adequately before application. Instil the whole quantity of product into bladder after having emptied it from residual urine. Instillation can be done by medical personnel, either by means of a probe, or by a syringe through urethra. Keep the preparation in bladder as long as possible, 30 minutes minimum. For urethral application, the preparation can be applied by means of a pipe or of a syringe.

Treatment duration and frequency will be decided by the doctor, although it is recommended that it should not be less than four instillations per week.

FOR PROFESSIONAL USE.

PROTECTIVE PREPARATION FOR VESICAL AND URETRAL MUCOSA

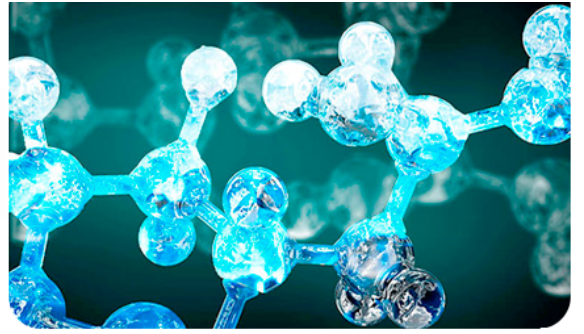

SHELF LIFE

3 years.

PRODUCT CATEGORY

Medical Device class IIa. GMDN [Global Medical Device Nomenclature]: 32616.