

## STERILE OCULAR WIPES

For periocular cleansing



### PRODUCT DESCRIPTION

The **STERILE OCULAR WIPES** are sterile cotton wipes, imbibed with a solution of vegetal extracts and hyaluronic acid, for the cleansing and maintaining of the periocular area.

Its hydrating and purifying action maintain the optimum conditions on this extremely sensitive area, and the calming and refreshing action reestablish the balance to the area.

### INDICATIONS

The **STERILE OCULAR WIPES** are effective for the hygiene of adults and children eyelashes, eyelids and surrounding areas.

Its use is especially effective if associated with the prescribed specialized therapy in case of pathologic ocular states as allergy, inflammation, phlogosis and other alteration of the normal periocular conditions.

### THE ACTIVE INGREDIENTS

The **STERILE OCULAR WIPES** is embedded in a solution containing ingredients specifically designed to work in synergy to achieve a high degree of cleanliness and comfort in the area:

- **HYALURONIC ACID**: a well-known active for its moisturizing and repairing properties. Get the ideal environment to the periocular area.

- **MIXTURE OF PLANT EXTRACTS**: with high calming, decongestant, purifying, refreshing and soothing effect. Among these ingredients we can find extracts of Chamomile, and Euphrasia.

Its formula is designed to allow use in children over 2 years and people with sensitive eyes and skin. There is also a variation of the formula to be suitable for babies.

### INSTRUCTIONS FOR USE

After washing hands thoroughly, open the sachet and take the wipe out.

Apply the wipe to eyelid area and rub gently, in order to have an adequate detersion and removal of ocular secretions and of eventual residues.

In case of particularly persistent secretions or lachrymal duct occlusions, it is useful to warm the sealed sachet for a few minutes in warm water [40°].

In case of phlogosis it is useful to cool the sealed sachet in the fridge for a few hours and then apply the open wipe on the eye, keeping it in place for a few minutes, as a cool compress.

## STERILE OCULAR WIPES

For periocular cleansing



Do not use the product if the sealed sachet is damaged or open.

Suitable for children over 2 years and persons with sensitive skin and eyes.

### PRODUCT CATEGORY

---

Medical Device Is.