

## COLD SORE PATCH



### PRODUCT DESCRIPTION

**COLD SORE PATCH** is a medical device that improves the healing process of cold sore.

This transparent patch helps preventing cross-contamination and reducing and facilitates the physiological recovery thanks to their occlusive and mechanical barrier action, mainly intended to keep under control the environment of the infected part.

It is a specially designed thin & round transparent film type of lip care patch, Acrylic pressure-sensitive adhesive. Useful and practical.

### MECHANISM OF ACTION

**COLD SORE PATCH** heals by mechanical action, providing virus static environment by isolated status [polyurethane barrier]. Physically effective on herpes by blocking oxygen.

The physical properties of the patch are intended to assist the healing process and to regulate the environment in terms of humidity and temperature, levels of oxygen and other gases, and pH value.

The patch is intended to keep under control the environment of the infected part, so also prevents the risk of secondary-infections and reduces the possibility of expansion of the infected area.

### INSTRUCTIONS FOR USE

Wash hands carefully before the application and be sure that the cold sore area is clean, dry, and free of cream to grant a good adhesion.

Take the patch out from its pouch. Pull the applicators with both hands until they separate. Each patch can remain in place on dry intact skin for up to 8-12 hours.

Recommended 2 patches per day. The time of application can vary depending on the cold sore symptoms. When patch is removed, peel off the patch slowly from the edges. Single use, not reusable.

### WARNINGS

For external use only. Do not apply in the mouth or mucosa. If any irritation and sensitivity occur, stop using it. Keep out of reaching of children. This patch does not contain any antiviral drug.

### PRODUCT CATEGORY

Medical Device class I.