

SCAR REDUCER PATCH



□ PRODUCT DESCRIPTION

The **scar reducer patch** is especially designed to help scars recover functionality and appearance and, therefore, to return to being normal skin.

A scar is a mark remaining in the tissue after a wound or sore has healed. When a wound occurs in the skin, a healing process is started that, if not performed correctly, will lead to a visible scar.

Under normal conditions, once the healing process is completed, the area's appearance should not diverge from the rest of the skin. However, it may happen that the result is not correct, whether due to insufficient or excessive scar tissue formation.

In both cases, the skin, besides losing its normal appearance, also loses functionality as the scar does not have the same elasticity as healthy skin and does not have as much mobility.

The **scar reducer patch** includes all prior experience in treating skin and wounds in order to fight scars in a convenient and efficient way as it is a 24-hour to 7 days application that is difficult to achieve with the usual creams and treatments.

The **scar reducer patch** is a hydro-active dressing, consequently, it does not soak the skin, it protects the scar from solar radiation, 50+SPF, and has a low risk of skin irritation.

□ INDICATIONS

The **scar reducer patch** is the best option for reducing hypertrophic, recent or old scars. This action is achieved thanks to the action of the adhesive in the dressing itself that joins and bonds the edges of the scar together and to its combination with low molecular weight Hyaluronic Acid, 50+ SPF.

It is also useful for keloids caused by burns, wounds or surgery.

Any treatment of scars should have two main objectives: to improve the appearance and functionality of the affected skin.

□ THE ACTIVE INGREDIENTS

Hyaluronic Acid is a natural component of the skin, necessary for repairing the dermis; a deficit of HA is associated with abnormal fibroblast arrangement and loss of hydration, key factors in scarring. Hyaluronic acid helps maintain the skin's optimum hydration conditions for repair.

Backed by our extensive scientific experience in the field of innovation and working together with expert dermatologists, when the art of knowing the skin acquired the utmost importance, we offer an innovative and high-quality product for caring for and treating visible scars.

SCAR REDUCER PATCH



INSTRUCTIONS FOR USE

Before applying the dressing, make sure that the area to be treated is clean and dry.

- 1) Cut the dressing to the desired size.
- 2) Separate the protective layer and place the dressing on the area ensuring, after applying mild and repeated pressure, that its edges are adhered to the skin.
- 3) The application should last 8 hours to 7 days, depending on requirements, then the patch should be replaced with a new one.
- 4) After a 4-week treatment, interrupt the application for 1 week, then start a new 4-week treatment if necessary and after consulting a doctor.
- 5) The patch is waterproof. In areas of high mobility, keep it in place with a bandage if necessary.

WARNINGS

External use. Use only on intact, clean and dry skin. Keep out of reach of children. Read the instructions for use carefully. Discontinue treatment in case of irritation or hypersensitivity to the product.

PRESENTATION

- 1 printed foil sachet containing 4 reusable patches that can be cut to size.
- Sachet size: 100 mm x 150 mm.
- Patch size: 50 mm x 70 mm.

PRODUCT CATEGORY

Medical Device class I.